

**PRODUCT SECURITY PATCH LIST  
FOR  
ANGIO WORKSTATION  
(2G984-0026EN)**

**CANON MEDICAL SYSTEMS CORPORATION**

## Trademarks

Windows® is a registered trademark of Microsoft Corporation in the United States and other countries.

This document may also include trademarks or registered trademarks of other companies.

Note that the trademark symbol "™" and the registered trademark symbol "®" may or may not be used in this document.

### **IMPORTANT!**

1. No part of this document may be copied or reprinted, in whole or in part, without written permission.
2. The contents of this document are subject to change without prior notice and without legal obligation.
3. The contents of this document are correct to the best of our knowledge. Please inform us of any ambiguous or erroneous descriptions, missing information, etc.

<b>REVISION RECORD</b>						
REV.	DATE (YYYY-MM)	REASON	/AUTHOR	PAGE CHANGED	SER. No.	DOC. PRODUCT.
INI.	2025-02		Saito	-----		NASU

Security Patch Package name

SPZX17-20084\_202501VL\_AWS90\_hotfix\_win10\_LTSC2019\_x64\_enu

Applicable Products for the Security Patch

XIDF-AWS801/B3	V9.0 SP0000* - V9.0 SP0005*
	V9.2 SP0000* - V9.2 SP0004*

Steps to Apply the Security Patch

Please follow the instruction of applying patches defined by each of the product Security Patch Package. For details, please contact the customer service for each system.

Verified Security Patch List

KB	MSRC	explanation(en-us)	URL/File Name	url
KB2538243_x86	MS11-025	Security Update for Microsoft Visual C++ 2008 Service Pack 1 Redistributable Package (KB2538243)	vcredist_x86.exe	<a href="https://catalog.s.download.windowsupdate.com/msdownload/update/software/secu/2011/05/vcredist_x86_470640aa4bb7db8e69196b5edb0010933569e98d.exe">https://catalog.s.download.windowsupdate.com/msdownload/update/software/secu/2011/05/vcredist_x86_470640aa4bb7db8e69196b5edb0010933569e98d.exe</a>
KB5005112	n/a	2021-08 Servicing Stack Update for Windows 10 Version 1809 for x64-based Systems (KB5005112)	windows10.0-kb5005112-x64_81d09dc6978520e1a6d44b3b15567667f83eba2c.msu	<a href="https://catalog.s.download.windowsupdate.com/d/msdownload/update/software/secu/2021/08/windows10.0-kb5005112-x64_81d09dc6978520e1a6d44b3b15567667f83eba2c.msu">https://catalog.s.download.windowsupdate.com/d/msdownload/update/software/secu/2021/08/windows10.0-kb5005112-x64_81d09dc6978520e1a6d44b3b15567667f83eba2c.msu</a>
KB5050008	n/a	2025-01 Cumulative Update for Windows 10 Version 1809 for x64-based Systems (KB5050008)	windows10.0-kb5050008-x64_80eb33a0afcef07b95b6d5925b174e6a189bb6c9.msu	<a href="https://catalog.s.download.windowsupdate.com/d/msdownload/update/software/secu/2025/01/windows10.0-kb5050008-x64_80eb33a0afcef07b95b6d5925b174e6a189bb6c9.msu">https://catalog.s.download.windowsupdate.com/d/msdownload/update/software/secu/2025/01/windows10.0-kb5050008-x64_80eb33a0afcef07b95b6d5925b174e6a189bb6c9.msu</a>
KB5049608	n/a	2025-01 Cumulative Update for .NET Framework 3.5 and 4.7.2 for Windows 10 Version 1809 for x64 (KB5049608)	windows10.0-kb5049608-x64_1a94518425035b7c11ccd554c89ae09752db511c.msu	<a href="https://catalog.s.download.windowsupdate.com/d/msdownload/update/software/secu/2024/12/windows10.0-kb5049608-x64_1a94518425035b7c11ccd554c89ae09752db511c.msu">https://catalog.s.download.windowsupdate.com/d/msdownload/update/software/secu/2024/12/windows10.0-kb5049608-x64_1a94518425035b7c11ccd554c89ae09752db511c.msu</a>
KB890830_v5.131	n/a	Windows Malicious Software Removal Tool x64 - v5.131 (KB890830)	windows-kb890830-x64-v5.131.exe	<a href="https://catalog.s.download.windowsupdate.com/c/msdownload/update/software/uprl/2025/01/windows-kb890830-x64-v5.131_b091e745124ba77a02629e887324f4802dc1e919.exe">https://catalog.s.download.windowsupdate.com/c/msdownload/update/software/uprl/2025/01/windows-kb890830-x64-v5.131_b091e745124ba77a02629e887324f4802dc1e919.exe</a>
KB2645410	n/a	Security Update for Microsoft Visual Studio 2010 Service Pack 1 (KB2645410)	kb2645410-x86.exe	<a href="https://catalog.s.download.windowsupdate.com/msdownload/update/software/secu/2012/02/vs10sp1-kb2645410-x86_0c3716da91c30a61d4c17747b03df2c7d5d07a08.exe">https://catalog.s.download.windowsupdate.com/msdownload/update/software/secu/2012/02/vs10sp1-kb2645410-x86_0c3716da91c30a61d4c17747b03df2c7d5d07a08.exe</a>
KB4505217	n/a	Security Update for SQL Server 2014 Service Pack 2 GDR (KB4505217)	kb4505217-x64.exe	<a href="https://catalog.s.download.windowsupdate.com/c/msdownload/update/software/secu/2019/06/sqlserver2014-kb4505217-x64_e622cdb8459a390f3d481ce9f90b357aa0851ca1.exe">https://catalog.s.download.windowsupdate.com/c/msdownload/update/software/secu/2019/06/sqlserver2014-kb4505217-x64_e622cdb8459a390f3d481ce9f90b357aa0851ca1.exe</a>
KB4052623_4.18.24090.11	n/a	Update for Microsoft Defender Antivirus antimalware platform - KB4052623 (Version 4.18.24090.11) - Current Channel (Broad)	updateplatform.exe	<a href="https://catalog.s.download.windowsupdate.com/d/msdownload/update/software/defu/2024/10/updateplatform.amd64fre_d3f6f8300855e56b8ed00da6dac55a3c4cbf8c20.exe">https://catalog.s.download.windowsupdate.com/d/msdownload/update/software/defu/2024/10/updateplatform.amd64fre_d3f6f8300855e56b8ed00da6dac55a3c4cbf8c20.exe</a>



LEGAL MANUFACTURER

**CANON MEDICAL SYSTEMS CORPORATION**

---

1385, SHIMOISHIGAMI, OTAWARA-SHI, TOCHIGI 324-8550, JAPAN